



Yes, you can.®

➤ Invacare® **Action<sup>3</sup> Junior**®

NEW

**Growing up with Action<sup>3</sup> Junior**®

Invacare® **Action<sup>3</sup> Junior**® is a lightweight, foldable paediatric wheelchair designed for children aged between 3 and 15 years. **Action<sup>3</sup> Junior**® has been developed to match the individual needs of the child and can grow as they grow. As a child's needs will change the **Action<sup>3</sup> Junior**® offers a large range of options to accompany the child in their development and to provide the necessary clinical support. The **Action<sup>3</sup> Junior**® is available with an outriggered frame and is particularly suitable for very active children. All models of **Action<sup>3</sup> Junior**® come in attractive, modern styles, designed to appeal to kids.



**JUNIOR**®



➤ **Grow with Invacare® Action<sup>3</sup> Junior®**

As children grow and develop, a key feature of a children's wheelchair is that it can be adjusted to accommodate the child's individual growth and needs. With the new Invacare® **Action<sup>3</sup> Junior®**, the width of the seat and backrest can be adjusted, as can the height of the arm and footrests.

➤ **An individual fit with extensive size combinations**

When choosing a paediatric wheelchair, a key factor is ensuring a perfect and comfortable fit for the child. The **Action<sup>3</sup> Junior®** comes in one of the most extensive ranges of size combinations, offering the right fit for all children, from toddlers to teenagers.

➤ **Optional design with increased stability for active children**

Active children who are likely to climb all over the wheelchair and be especially energetic when propelling the chair forward will benefit from the outriggered version of the **Action<sup>3</sup> Junior®**. This model features rigged out front castors to prevent the child from tipping forward when climbing up or over the footplates.

➤ **Easy to transport, light in weight**

Thanks to the lightweight design, folding backrest, quick release wheels and detachable arm- and leg rests, the **Action<sup>3</sup> Junior®** is very suitable for transportation and storage.



## Options and Features



### Seat-width adjustment

The seat width of the chassis can be adjusted with a new telescopic crossbar for four fixed-seat positions. (**Action<sup>3</sup>** - Small: 20,5 to 28 cm, **Action<sup>3</sup>** - Large: 30,5 to 38 cm.)



### Upholstery adjustment

As the chassis is made wider, so the upholstery on the back and on the seat of the chair can be made wider with Velcro flaps.



### Adjustable seat height and balance

For improved driving performance and different height options, the rear wheel can be mounted in 20 different positions. Wheels can be positioned to 'active', 'standard', 'comfort' or 'amputee' rear wheel position to suit the child's different needs.



### Table tray

We also offer a swing-away tray which can easily be folded away.



### Backrest types

On top of the fixed backrest (0 or 7 degree),

*Invacare<sup>®</sup> Action<sup>3</sup> Junior<sup>®</sup>* offers 4 additional types of backrests to suit user's needs:

- ① Gas spring reclining (0/30 degree)
- ② Ratchet reclining (0/30 degree)
- ③ Folding backrest with fixed push handles
- ④ Folding backrest with height adjustable push handles



### Spoke guard

Three different types of spoke guards are offered to match the different wheelchair colours and tastes.



### Armrests


Removable and height adjustable desk and junior armrests as well as skirt- and mudguards are available.




### Outrigged Front Frame

To prevent the child from tipping forward when climbing up or down the foot plates, an outrigger front frame is available as an option.


## Technical data\*

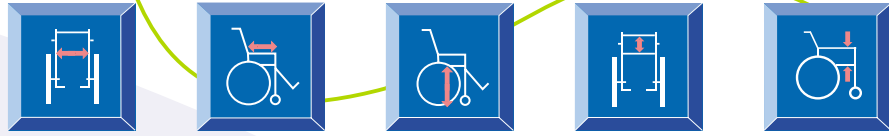

<b>Action<sup>3</sup> Junior® - Small</b>	20,5 / 23 / 25,5 / 28	20-25 / 25-30 / 30-35	41 / 43,5 / 46	37 / 40 / 43 <sup>①</sup>	15 - 25 <sup>②</sup> / 18 -28 <sup>③</sup>
<b>Action<sup>3</sup> Junior® - Large</b>	30,5 / 33 / 35,5 / 38	27,5-32,5 / 32,5-37,5	41 / 43,5 / 46	37 / 40 / 43 <sup>①</sup>	15 - 25 <sup>②</sup> / 18 -28 <sup>③</sup>
<b>Action<sup>3</sup> Junior® Outrigged - Small</b>	20,5 / 23 / 25,5 / 28	20-25 / 25-30 / 30-35	41 / 43,5 / 46	37 / 40 / 43 <sup>①</sup>	15 - 25 <sup>②</sup> / 18 -28 <sup>③</sup>
<b>Action<sup>3</sup> Junior® Outrigged - Large</b>	30,5 / 33 / 35,5 / 38	27,5-32,5 / 32,5-37,5	41 / 43,5 / 46	37 / 40 / 43 <sup>①</sup>	15 - 25 <sup>②</sup> / 18 -28 <sup>③</sup>

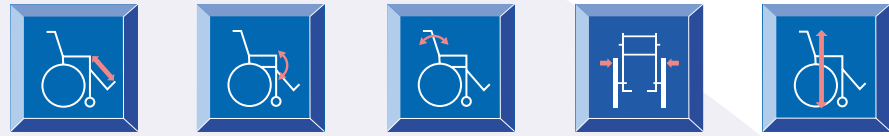

<b>Action<sup>3</sup> Junior® - Small</b>	13-27 <sup>④</sup> / 22-32 <sup>⑤</sup> / 30-45 <sup>⑥</sup>	3° / ±1°	0° / 7° and 0°-30° <sup>⑦</sup>	SW +19 / +17,5 <sup>⑧</sup>	87 - 113
<b>Action<sup>3</sup> Junior® - Large</b>	13-27 <sup>④</sup> / 22-32 <sup>⑤</sup> / 30-45 <sup>⑥</sup>	3° / ±1°	0° / 7° and 0°-30° <sup>⑦</sup>	SW +19 / +17,5 <sup>⑧</sup>	87 - 113
<b>Action<sup>3</sup> Junior® Outrigged - Small</b>	8,5 - 20,5	3° / ±1°	0° / 7° and 0°-30° <sup>⑦</sup>	SW + 27	87 - 113
<b>Action<sup>3</sup> Junior® Outrigged - Large</b>	8,5 - 20,5	3° / ±1°	0° / 7° and 0°-30° <sup>⑦</sup>	SW + 27	87 - 113

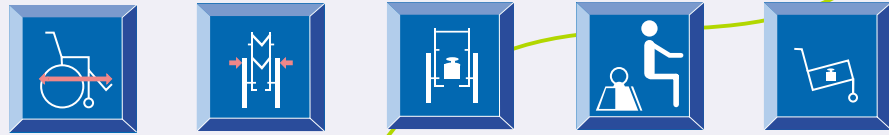

<b>Action<sup>3</sup> Junior® - Small</b>	67 - 97	32,5	12,3	80	7,5
<b>Action<sup>3</sup> Junior® - Large</b>	67 - 97	35	15	80	9
<b>Action<sup>3</sup> Junior® Outrigged - Small</b>	86 - 91	39	11,6	80	7,4
<b>Action<sup>3</sup> Junior® Outrigged - Large</b>	86 - 91	39	13,3	80	8,9

- ① Please refer to your prescription form for possible adjustment of upholstery and pushhandles
- ② Desk
- ③ Junior
- ④ 90 degree hanger

- ⑤ 80 degree short hanger
- ⑥ 80 degree long hanger
- ⑦ Recliner
- ⑧ Transit Version

\* measures in cm, weights in kilo

## Frame Colours



Tangerine Orange



Dark Red



Sapphire Blue



Pink



Anthracite Black



Black Nylon

## Upholstery

All information quoted is believed to be correct at time of print. Invacare® reserves the right to alter product specifications without prior consultation.



INVACARE® Limited  
 South Road · Bridgend Industrial Estate  
 BRIDGEND CF31 3PY  
 Tel. +44 (0) 1656 647327 · Fax +44 (0) 1656 649016  
 E-mail: uk@invacare.com · [www.invacare.co.uk](http://www.invacare.co.uk)

INVACARE® Ireland Ltd  
 Unit 5 · Seatown Business Campus  
 Seatown Road · SWORDS · County Dublin  
 Tel. (353) 181 107 84 · Fax (353) 181 107 85  
 Email: eire@invacare.com · [www.invacare.ie](http://www.invacare.ie)



Action<sup>3</sup> Junior - Part-No. 1504190

© Invacare® International SARL 2007. All rights reserved.

